
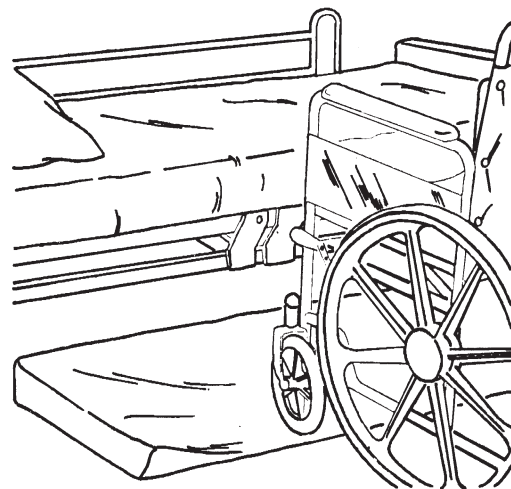


## Roll-On Bedside Mat

AL #	Description	Size	Unit
AL-62424	Roll-On Bedside Mat	68" x 26" x 1.5"	Each
AL-73233	Roll-On Bedside Mat	68" x 36" x 1.5"	Each

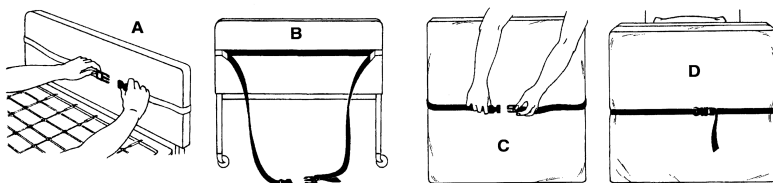

### PURPOSE

Placed alongside a bed, the Roll-On Bedside Mat reduces the impact of a fall from bed. The tapered edges are designed for easy wheelchair roll-on and are safer for residents who might stumble on standard floor mats.

### INSTRUCTIONS

Place Roll-On Bedside Mat on the floor next to the bed. Two mats will be required if side rails on both sides of the bed are lowered. Important: Place the Roll-On Bedside Mat on the floor with the label reading THIS SIDE UP facing up.

### Important Notices

1. It is the responsibility of each facility's appropriate medical professionals to determine if the Roll-On Bedside Mat is suitable as an injury-prevention device for a specific patient.
2. To prevent staff members from stumbling, make certain that all staff members, especially those working at night, are aware that the Roll-On Bedside Mat has been placed alongside a bed.

### Cleaning Instructions

Comply with OSHA regulations and CDC recommendations relating to blood borne pathogens. Follow accepted precautions. Use an EPA-registered disinfectant according to the manufacturer's instructions when performing disinfecting procedures.

### Disinfection

1. Before disinfection, clean the cover surface using a damp cloth or sponge and mild cleanser.
2. Disinfect the cover surface using an appropriate EPA-registered disinfectant and follow manufacturer's instructions. Do not saturate the cover or allow cleaning solutions to pool on the cover surface.

### Infection

Inspect the cover before each use. Do not use if the cover has cuts, rips, cracks, or tears.