

PRIVACY SYSTEMS/SOFT GOODS

CUBICLE TRACK CARRIERS

Many styles of cubicle track carriers are offered for use with Whisper Cube, Ultra Cube, Optitrac and Econo Cube track systems. Load capacity test data reports available upon request. Note: Carrier heights and wheel heights are approximate measurements only. Please request samples prior to ordering.

OPTITRAC CARRIERS

Three styles of carriers offered that glide effortlessly through track.



BALL & CHAIN CARRIER AL-68207

Traditional chrome plated chain and hook.

Carrier height: 2 1/2"

Wheel height: 1/2"

Peak Load, lb(f): 46.9



SPOOL CARRIER

Conventional spool and hook.

Carrier height: 2 5/8"

Wheel height: 11/16"

Peak Load, lb(f): 53.1



POP OUT CARRIER AL-70761

The hook portion releases, allowing the curtain to fall. Use for safety in psychiatric care areas and correctional facilities. Disengages at ± 10 lbs. of pressure.

Carrier height: 2 1/2"

Wheel height: 5/8"

Peak Load, lb(f): 20.4

WHISPER CUBE AND ULTRA CUBE CANTED WHEEL CARRIERS

Three styles of carriers offered with patented canted wheels that glide effortlessly through track. More material makes carriers last longer than less substantial, traditional carriers.



SOLID HOOK AL-50178

Withstands years of constant abuse.

Carrier height: 2 3/8"

Wheel height: 5/8"

Peak Load, lb(f): 53.3



BALL & CHAIN AL-50180

Traditional chrome plated chain and hook.

Carrier height: 4"

Wheel height: 5/8"

Peak Load, lb(f): 46.9



POP OUT AL-50182

The hook portion releases, allowing the curtain to fall. Use for safety in psychiatric care areas and correctional facilities. Disengages at ± 10 lbs. of pressure.

Carrier height: 2 7/8"

Wheel height: 5/8"

Peak Load, lb(f): 20.3

ECONO CUBE CARRIER

One style of carrier designed specifically for this system.



CARRIER AL-50181

Available in sets of 10 for easier installation.

Carrier height: 2"

Wheel height: 5/16"

Peak Load, lb(f): 51.7

Clickeze Clean-up and Maintenance:

Cubicle Track, Curtains, and Soft Goods

Cubicle Tracks

Cubicle tracks require very little maintenance. If carriers do not move smoothly, check the track for binding. If the track is clear of debris, spray the inside of the track with a light coating of silicone spray. Be careful not to spray the curtains and wipe any excess spray from the track.

Standard Cubicle Curtains, Cubicle Curtain Mesh or Standard Fabric Only

1. Wash Cycle: Wash as any standard colored fabric, with water temperature not to exceed 160°F. Detergents: Use a synthetic detergent. DO NOT use a caustic soap or bleach. Do Not add softener.
2. Extraction Cycle: Fabric requires no more than 2-4 minutes extracting, depending on size of load and equipment used.
3. Drying Cycle: Tumble dry for 3-5 minutes on the synthetic cycle. DO NOT overload the dryer. Excessive heat will decrease the permanent press feature of this fabric. Remove immediately from the dryer after the cycle is complete.
4. DO NOT put through a mangle.

Shield by Panaz® Fabric or Cubicle Curtains

1. Wash Cycle: Spot clean with damp sponge/cloth with general detergent or germicidal cleaner. It is recommended to spot test the cleaner if the effect on polyester fabric is not known. Fabric may be laundered by steam cleaning or commercial washing where water temperature does not exceed 160°F. Commercial detergents will not harm Shield properties. Do Not add fabric softeners. Repeated washing does not destroy inherent properties of the Shield by Panaz® fabric.
2. Extraction Cycle: Extraction depends on the size of the load and equipment used. Fabric requires 2-4 minutes of extraction.
3. Drying Cycle: DO NOT overload the dryer. Use synthetic (also low temp or knit cycle) heat setting and cycle for 3-5 minutes. Remove and fold immediately to retain ironed appearance.
4. Finishing: If fabric is folded when drying cycle is completed, no ironing is required. Occasionally light touch-up with hand iron (275°), if desired.

Draperies

Draperies should be cleaned as seldom as possible with dusting or vacuuming done on an "as needed" basis. Frequent or repeated cleanings may remove some of the "life" from a drape eventually yielding limpness to the material. Dry cleaning is the preferred method for cleaning.

Vinyl Shower Curtain Washing Instructions

RECOMMENDED

1. Sponge with warm mild soap & rinse with clear water.

MACHINE WASHING

1. Relatively cool water temperature not to exceed 120°F.
2. Use mild soap solution, do not use commercial detergent/ chlorine bleach.
3. Air dry at low temperature.


Single Side Printed Bedspreads Washing Recommendations

1. No fabric softener or bleach is used.
2. Machine wash or tumble dry low not to exceed 120°.
3. Do not wash polyester bedspreads with any cotton items or in a machine which has cotton lint residue from previous washings.
4. Wash with a mild detergent promptly after spotting so stains will release properly.


Bedspreads

1. Wash separately (do not combine with other products)
2. Wash darks separately
3. Machine wash (120° F - 160° F)
4. Tumble dry - medium - remove promptly
5. Use steam iron - low setting
6. DO NOT DRY CLEAN
2-3% shrinkage anticipated in washing cycle

AL-50178
SOLID HOOK CARRIER
COLOR: WHITE or BLACK

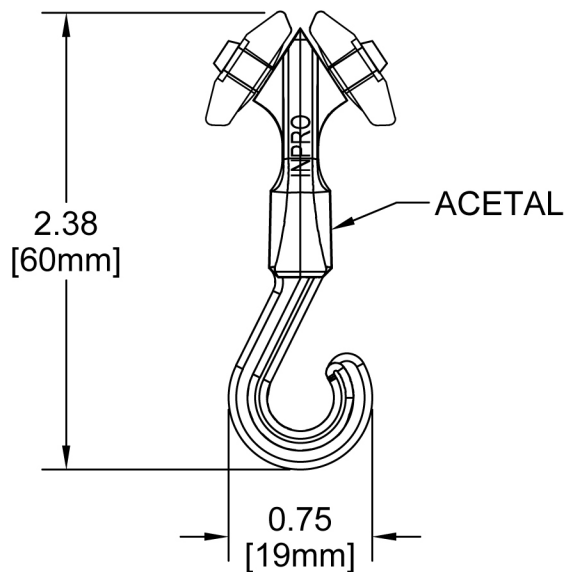

AL-50182
POP-OUT CARRIER
COLOR: WHITE

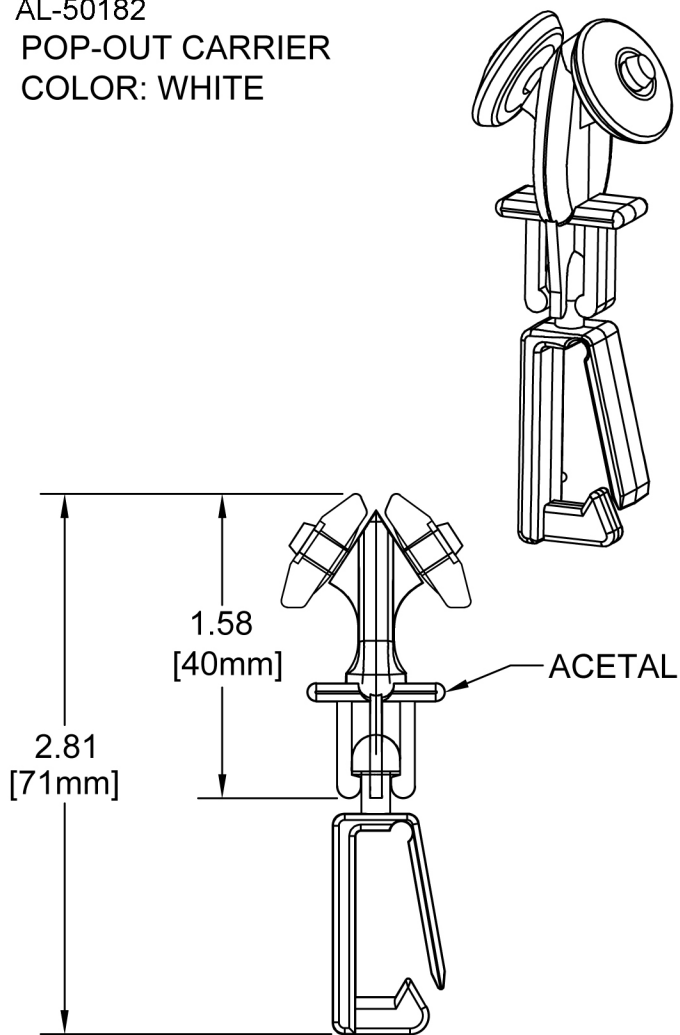

AL-50180
BALL & CHAIN CARRIER
COLOR: WHITE

