

MadaCide-FD

GERMICIDAL SOLUTION

DISINFECTANT / CLEANER / DEODORIZER



AL-54897
32 oz. Spray Bottle

AL-54898
1 Gallon (4/case)

BROAD SPECTRUM EFFECTIVENESS

- ✓ Bactericidal
- ✓ Tuberculocidal
- ✓ Fungicidal
- ✓ Virucidal

FEATURES / BENEFITS

- ✓ Ready to Use
- ✓ Mild Scent
- ✓ Low Alcohol

ORDERING INFORMATION

AL-54897	32 oz. Spray Bottle	(ea)
AL-54898	1 Gallon	(4/case)
AL-69727	2.5 Gallon	(ea)
AL-54899	5 Gallon	(ea)
AL-69728	55 Gallon	(ea)
Call	Pump for 7024	(ea)



6851 High Grove Blvd., Burr Ridge, IL 60527 USA

800.323.4282 | www.alcosales.com

MadaCide-FD

GERMICIDAL SOLUTION

MadaCide-FD is a general purpose cleaner, deodorizer and disinfectant to be used where control of cross-contamination from treated surfaces is of prime importance.

FOR USE ON ALL HARD, NON-POROUS SURFACES INCLUDING:

- Chrome
- Formica® Brand
- Glass
- Painted Surfaces
- Polycarbonate
- Polyvinylchloride
- Polypropylene
- Polystyrene
- Plexiglas®
- Glazed Porcelain
- Stainless Steel
- Vinyl

AREAS OF USE:

Counters/Countertops, Operating Room Lights/Tables/Surfaces, **Dental** Instruments/Implements/Surfaces, **Medical** Instruments/Implements/Surfaces, **MadaJets, Ambulance** Equipment/Surfaces, **Laboratory** Equipment/Surfaces, **Respiratory Therapy** Equipment, **Respirators**/Face Pieces, **CPR Training** Mannequins, **Nail Care** Implements, **Exercise** Equipment/Benches/Mats, **Tanning Beds**, Hardhats, **Headphones**,

ORGANISMS

KILL TIME

Herpes Simplex II, HIV-I (Aids Virus), Influenza Strain A2/HK	30 sec
Rhinovirus, Trichophyton mentagrophytes	2 min
Methicillin Resistant Staphylococcus Aureus (MRSA)	3 min
Hepatitis B & C Virus (HBV, HBC), Pseudomonas aeruginosa, Salmonella enterica, Staphylococcus aureus	3 min
Aspergillus niger, Candida albicans, Vancomycin-resistant Enterococcus faecalis (VRE)	5 min
Mycobacterium bovis BCG (Tuberculosis)	6 min

ULTRASONIC CLEANING:

Thoroughly pre-rinse instruments under running water to remove visible gross debris. Using 1 ounce of this product per liter of water in ultrasonic unit, immerse instruments into solution and activate ultrasonic unit for 5 minutes (or longer if necessary). Remove and rinse instruments. Follow with appropriate cleaning and disinfection process. Change solution as needed when it becomes diluted or visibly soiled.



6851 High Grove Blvd., Burr Ridge, IL 60527 USA

800.323.4282

| www.alcosales.com